

Lyof foam® Max




A HIGHLY ABSORBENT POLYURETHANE FOAM DRESSING

Lyof foam® Max is a highly absorbent dressing. It consists of a polyurethane foam and a moisture proof backing film with high permeability to allow for effective exudate management of moderate to highly exuding wounds.

Polyurethane backing film


- Moisture vapor permeable
- Moisture Proof
- Provides bacterial and viral barrier

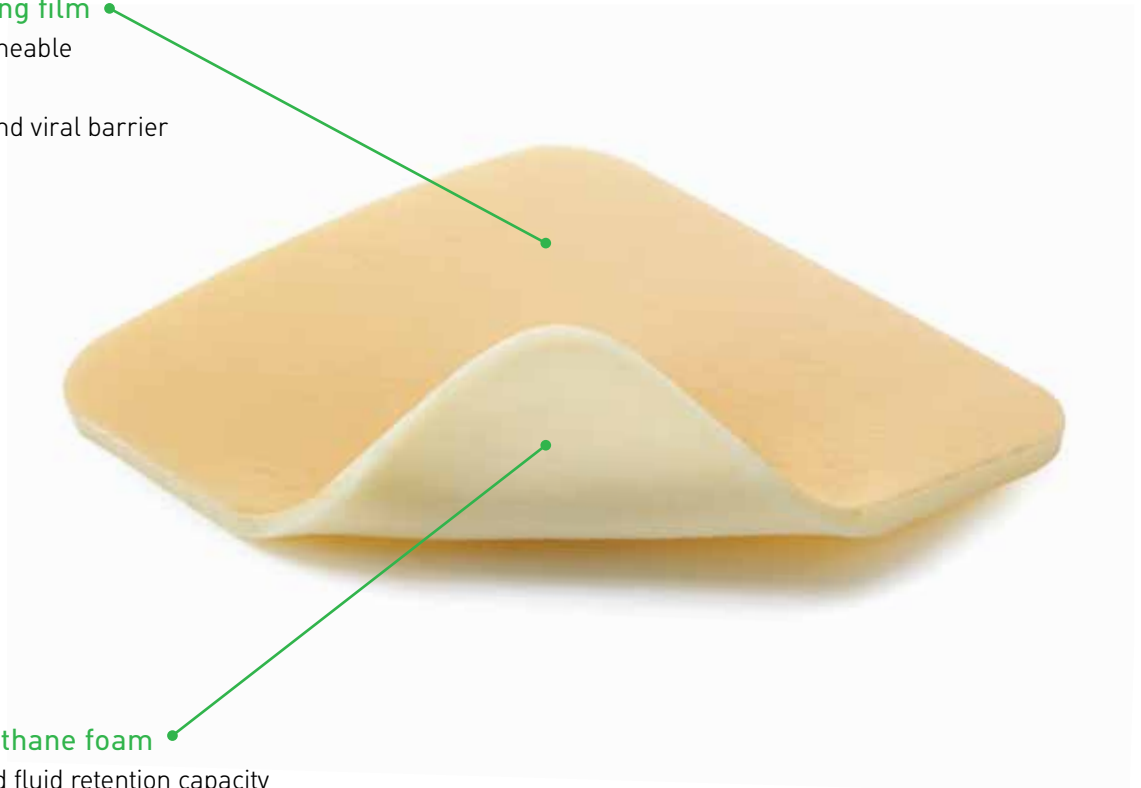

Hydrophilic polyurethane foam

- High absorbency and fluid retention capacity
- May be used under compression
- Soft, conformable and easy to apply and remove


How to use Lyofoam® Max

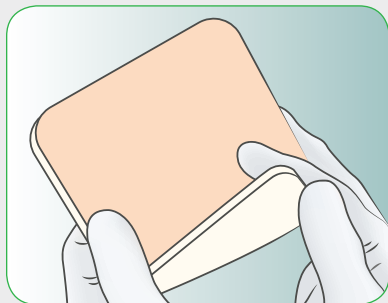

Clean the wound area and select the appropriate size of **Lyofoam® Max**. The dressing should overlap the wound margin by about 2 cm.



Place the foam side directly onto the wound surface, with the shiny side of the dressing facing up.

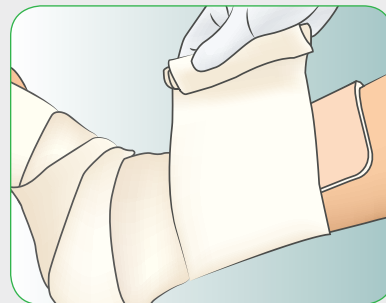

Secure **Lyofoam® Max** with an appropriate bandage such as **Tubifast®**, **Tubigrip®** or a fixation tape such as **Mefix®**.

Key benefits of Lyofoam® Max

- High absorbency and fluid retention capacity¹
- Reduced risk of maceration, less potential for leakage and longer wear-time¹
- Moisture proof backing film acts as a barrier to bacteria and virus penetration
- Works under compression
- Easy to apply and remove
- Soft and conformable and provides comfort to patients
- Non-sensitizing
- Wide range of sizes available for different wound types

Areas of use

Lyofoam® Max is designed for the management of moderate to heavily exuding wounds such as leg ulcers and pressure ulcers.

Through an excellent exudate management, **Lyofoam® Max** creates a moist wound environment optimal wound for healing.

Frequency of change


Lyofoam® Max may be left in place for several days depending on the nature and condition of the wound and the level of exudate.

References


1. SMTL Report No. 11/3571/6
2. SMTL Report No. 11/3726/1
3. SMTL Report No. 11/3726/2
4. SMTL Report No. 11/3571/2
5. SMTL Report No. 11/3571/3

Lyofoam® Max - even better performance

Better absorbency¹

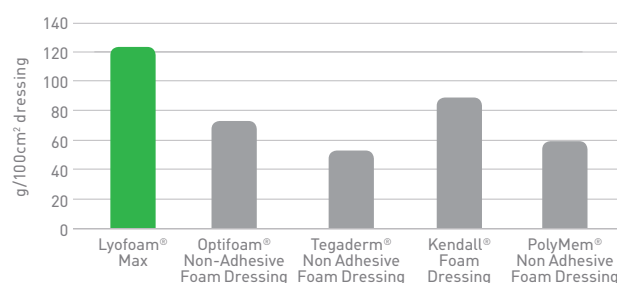

Better fluid handling¹

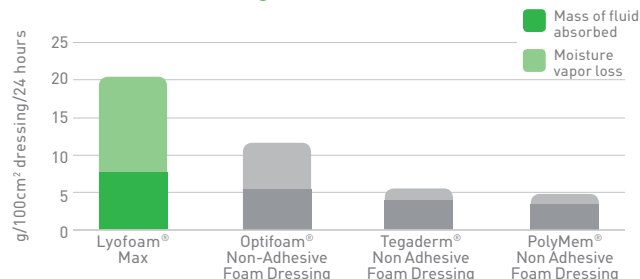

Lyofoam® Max Ordering Information†

Product Code	Size	Pieces/box	HCPCS
603200	3" x 3.4" (7.5 x 8.5 cm)	10	A6209
603201	4" x 4" (10 x 10 cm)	10	A6209
603203	4" x 8" (10 x 20 cm)	10	A6210
603204	6" x 6" (15 x 15 cm)	10	A6210
603205	6" x 8" (15 x 15 cm)	10	A6210
603206	8" x 8" (20 x 20 cm)	10	A6211

† Packaged sterile in single packs



5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 800.882.4582 | www.molnlycke.us | www.mhcwoundcare.com

The Mölnlycke Health Care, Lyofoam, Tubifast, Tubigrip, and Mefix, names and respective logos are registered trademarks of Mölnlycke Health Care AB. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2011. Mölnlycke Health Care AB. 1.800.882.4582.

PermaFoam, ActivHeal, Kendall, and PolyMem are registered trademarks of Hartmann-Conco, Inc., Advanced Medical Solutions Limited, Covidien AG, and Ferris Corporation respectively. Optifoam is from Medline Industries, Inc. Tegaderm is from 3M Company.

MHC-2011-0455 Rev1 - 270370.12.11