

Mack's® Sound Asleep™ Soft Foam Earplugs
12 Pair

Indications

Reduces noise

Directions

1. With clean hands, roll plug into a tight, crease-free cylinder.
2. With opposite hand, reach over head and pull up and back on outer ear to straighten ear canal.
3. Quickly insert earplug well into ear canal and hold in until it expands for a comfortable, secure fit.
4. Discard after several uses or as soon as dirt or wax accumulates on earplugs.

Ingredients

Urethane foam

Warnings & Precautions

WARNING: Remove earplug slowly with twisting motion to gradually break the seal. Rapid removal may damage eardrum.

WARNING: Keep away from infants, small children and pets. These earplugs are non-toxic, but may interfere with breathing if caught in windpipe, which could lead to serious injury or death.

WARNING: Improper fitting will reduce effectiveness and could result in hearing loss or injury. Do not use to seal out water. Use Mack's Pillow Soft Silicone Earplugs, Mack's AquaBlock or Mack's Ear Seals for water protection.

Company Information
McKeon Products, Inc.
25460 Guenther, Warren, MI 48091
(586) 427-7560
www.MacksEarplugs.com

Copyright
© 2010 McKeon Products, Inc.

