

**Mack's® Hear Plugs® High Fidelity Earplugs**  
**1 Pair**

**Indications**

Reduce harmful while hearing the full range of sound, music and communication

**Directions**

1. Reach over head and gently pull ear up and back on outer ear to straighten ear canal.
2. With opposite hand, grasp earplug stem and insert by pressing forward in a circular motion to form airtight seal.
3. Wear the plugs the entire duration of the loud event.

- Mack's High Fidelity Hear Plugs use a unique patented tuned resonator and acoustic resistor to reduce all sound evenly across the frequency range.

- Speech and music stay clear, just quieter, like turning down the volume.

- For optimal effectiveness, please ensure that the earplugs are fitted and maintained in accordance with the instructions.

- Before use, check for signs of wear. If earplugs become hard, torn or deformed, they should be replaced.

**Ingredients**

Super-soft silicone

**Warnings & Precautions**

**WARNING:** Remove slowly with twisting motion to gradually break the seal.

**WARNING:** Keep away from infants and small children. These earplugs are non-toxic, but may interfere with breathing if caught in windpipe, which could lead to serious injury or death.

Caution: Use only as directed.

**Cleaning Instructions**

After use, wipe earplugs clean with damp cloth. Do not immerse in water. Alcohol is not recommended.

**Company Information**

McKeon Products, Inc.  
25460 Guenther, Warren, MI 48091  
(586) 427-7560  
[www.MacksEarplugs.com](http://www.MacksEarplugs.com)

**Copyright**

© 2010 McKeon Products, Inc.

