

**Dreamgirl Sleep Mask**  
**1 Sleep Mask – 1 Pair Soft Foam Earplugs**

**Indications**

Blocks the transmission of light to the eyes.

**Directions**

Use velcro fastener to tighten straps around head for a comfortable, secure fit. Make sure outside strap uniformly covers inside strap to help prevent uncovered velcro from coming in contact with hair.

**Washing Instructions**

Machine wash on gentle cycle or by hand. Air dry

**Soft Foam Earplugs**

**Indications**

Reduces noise

**Directions**

1. With clean hands, roll plug into a tight, crease-free cylinder.
2. With opposite hand, reach over head and pull up and back on outer ear to straighten ear canal.
3. Quickly insert earplug well into ear canal and hold in until it expands for a comfortable, secure fit.
4. **WARNING:** Improper fitting will reduce effectiveness and could result in hearing loss or injury.

Discard after several uses or as soon as dirt or wax accumulates on earplugs.

**Ingredients**

Urethane foam

**Warnings & Precautions**

**WARNING:** Remove earplug slowly with twisting motion to gradually break the seal. Rapid removal may damage eardrum.

**WARNING:** Keep away from infants and small children. These earplugs are non-toxic, but may interfere with breathing if caught in windpipe, which could lead to serious injury or death.

**Company Information**

McKeon Products, Inc.  
25460 Guenther, Warren, MI 48091  
(586) 427-7560  
[www.MacksEarplugs.com](http://www.MacksEarplugs.com)

**Copyright**

© 2010 McKeon Products, Inc.

