



Mack's® Pillow Soft Earplugs®
2 Pair

Indications

Seals out water and reduces noise.

Directions

DO NOT INSERT INTO EAR CANAL. COVER ONLY.

1. Make sure hands and ear opening are clean and dry. Shape whole plug into a ball.
2. Place plug over ear opening and flatten to form airtight seal. Avoid hair.
3. Do not insert into ear canal!
4. To loosen and remove, press up behind ear. Discard when soiled or no longer sticky (up to 5 uses). For best results when flying, wear earplugs entire flight. Do not cut, break apart or elongate earplugs.

Ingredients

Moldable, non-irritating silicone rubber

Warnings & Precautions

WARNING: Use whole earplug. Cover only. Do not force plug into ear canal. Do not cut, break apart or elongate earplugs. If misused by failure to use as directed, earplug may become lodged in ear, requiring removal by an Ear, Nose and Throat Doctor ONLY!

WARNING: Adult supervision required for children under 15. Keep away from infants and small children when not in use. These plugs are non-toxic but may interfere with breathing if caught in windpipe, which could lead to serious injury or death. Do not use for scuba diving or at depths over 10 feet. Use only as directed.

CAUTION: Lubricants and solvents can degrade this product. Discard if earplugs come in contact with lubricants, solvents or other chemicals. May be use in chlorinated water.